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SX0008R00P01-0820
#VER_ST_0820/BL-01

[EN] Consumer Information

A full contact flexible wound dressing for the prevention and treatment of radiation dermatitis.

StrataXRT is a semi-occlusive, self-drying and transparent gel.

When used as directed StrataXRT dries to form a protective layer that is gas permeable and waterproof which hydrates and protects compromised skin areas and superficial wounds from chemical and microbial invasion.

StrataXRT may be directly applied to radiation dermatitis, compromised skin surfaces, superficial wounds.

How StrataXRT works

StrataXRT dries to form a thin, flexible wound dressing which hydrates and protects affected areas.

StrataXRT has no measurable pH value so does not affect the fragile acid mantle of the skin.

StrataXRT helps to promote a moist wound healing environment for widespread superficial wounds and burns.

StrataXRT is a self-drying, non-sticky, transparent, gel formulation that lightly bonds to the most superficial damaged skin layer.

StrataXRT gel is bacteriostatic and inert.

This moist wound healing environment promotes faster re-epithelialization and significantly reduces the skin's acute inflammatory response seen with both dry and wet desquamation from radiation dosing.

StrataXRT is ideal for large surface areas and contoured skin like the breasts, face, back and pelvic area, as well as joints and hairy areas without the need for shaving.

Intended use

StrataXRT is used to relieve low grade inflammatory changes such as dry, itching, flaking, peeling and irritated skin.

StrataXRT was developed for use on all types of radiation dermatitis, toxic and compromised skin, superficial wounds, including those resulting from radiation therapy.

StrataXRT may be applied to superficial wounds and areas where skin integrity has been compromised.

StrataXRT may also be used in conjunction with other adjunctive treatments to improve overall results.

StrataXRT contains no alcohols, parabens or fragrances and is transparent and odorless.

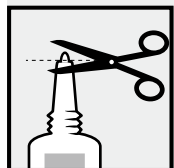
StrataXRT can be used with or without a secondary protective dressing.

StrataXRT is suitable for children. StrataXRT is intended for single patient use.

Directions for use

On first use 20 g tubes remove the cap, cut the nozzle tip of the tube and close with the cap after use.

On 50 g tubes remove the cap, then the protective seal and close with the cap after use.

Ensure that the affected skin superficial wound is clean and dry.

Gently pat dry as much excess exudate or wound fluid from the area as possible prior to gel application.

Apply a very thin layer of StrataXRT directly to the affected area and allow the gel to dry. StrataXRT should be applied once or twice daily to affected areas or as advised by your physician.

Once dry, StrataXRT may be covered by sunscreen, cosmetics and clothing.

For best results StrataXRT should be maintained in continuous contact with the skin (24 hours a day / 7 days a week).

When applied correctly to exposed areas, StrataXRT should be dry in 5–6 minutes.

If it takes longer to dry you have probably applied too much.

Gently remove the excess with a clean tissue or gauze and allow the drying process to continue.

Recommended duration of treatment

StrataXRT is recommended as prophylaxis following the initial radiation dose and should continue to be applied for a minimum of 60–90 days (24 hours a day / 7 days a week) post radiation therapy, or until no further improvement is seen.

For chronic radiation dermatitis, continued use is recommended until no further improvement is seen.

How much StrataXRT do I need?

StrataXRT gel is a unique formulation that requires substantially less product per application than typical moisturizing creams or barrier ointments.

- **StrataXRT 20g** is enough to treat an area of 12x15 cm, twice per day for over 30 days.
- **StrataXRT 50g** is enough to treat an area of 12x15 cm, twice per day for over 75 days.

Caution

- For external use only.
- StrataXRT should not be placed in contact with the eyes.
- StrataXRT should not be applied over topical medications unless advised by your physician.
- StrataXRT may stain clothing if not completely dry. If staining occurs, dry cleaning should be able to remove it without damaging the fabric.
- For correct storage please reclose the tube tightly with the cap.
- Should your radiation dermatitis show signs of infection or failure to heal after 7 days of initial product application, consult your physician.
- If irritation occurs, discontinue use and consult your physician.
- Not suitable for highly exudative wounds, tunneling wounds or 3rd degree burns.
- Keep out of the reach of children.
- Do not use after the expiration (EXP) date printed on the tube. The expiration (EXP) date does not change once the tube has been opened.
- Do not use if the tube is damaged.

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Date of preparation of this information 08/2020

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Stratpharma AG